

Small Animal Benchtop Scales – Instruction Manual



**Please read this instruction booklet carefully
before using this product.**



GVP Pty Ltd

Product Code

GVPT-E-WOSFL-4010

Product Specifications

Model Number	KN-B1H
Power Supply	2 x AAA batteries
Measurement Range	min. 50g to max. 40kg
Precision	10g graduation
Error	± 0.3%
Display	Blue LCD background with white value display
Measurement Units	Kg/lb/oz
Power saving mode	Automatic power off after 60 seconds

Assembly guide

1. The scales come in three separate pieces. Remove all packaging and place all pieces on a clean, flat surface.
2. Install the 2 x AAA batteries provided in the underside of the base.
3. Clip the two halves of the curved tray together and place on top of the base.
4. Ensure the scales are placed on a level table surface.
5. Press the 'ON/OFF' button to start the electronic scale.
6. To zero the scales, hold the 'TARE' button until '0' is shown on the LCD display screen.
7. The 'UNIT' button can be used to alter the units in which the results are being displayed.
8. The scales are now ready for weighing the patient.



Button Descriptions

ON/OFF – This button will start up and shut down the electronic scale.

TARE – This button will zero the scale and display a '0' value when the scale is ready for use.

HOLD – This button will hold the numeric value on the screen once the patient has been removed. Only press this button once the weight value is stable (not fluctuating). To cancel the holding of the value, press the HOLD button a second time.

UNIT – This button will allow the user to select in which units the measurement will be displayed. Values can be shown in kilograms (kg), pounds (lb), and ounces (oz).

Important Information

1. Ensure the scales are placed on a clean, flat surface before every use.
2. Do not use the scales in extreme hot or cold temperatures.
3. Do not get the base of the scales wet or submerge it in water.
4. Do not tap the screen or the buttons in a vigorous manner.
5. When moving the scales, avoid dropping the base of the scales as they contain high precision electronics and can damage easily.
6. When cleaning the scales, only use a soft, clean cloth. Do not use chemical solutions.
7. Remove batteries if the scales are not being used for a long period of time.
8. Replace batteries when 'LO' is displayed on the LCD screen.
9. If the display shows 'ERR', this means the scales have been overloaded and the patient should be removed immediately.



Supplier Contact Information

GVP – General Vet Products

PO Box 417

Fairy Meadow NSW 2519

T: 1300 VET GET / 1300 838 438

E: sales@generalvetproducts.com.au

W: www.generalvetproducts.com.au



GVP Pty Ltd