
# Trimmer StyleMidi (OC20)

**Bedienungsanleitung**  
**Instruction manual**  
**Mode d'emploi**  
**Manual de instrucciones**  
**Instruções de manejo**  
**Manuale di istruzioni**  
**Gebruiksaanwijzing**  
**Betjeningsvejledning**  
**Brugsanvisning**  
**Käyttöopas**  
**Bruksanvisning**  
**Οδηγία χρήσης**  
**Инструкция по эксплуатации**  
**取扱説明書**  
**사용 설명서**  
**تعليمات الاستخدام**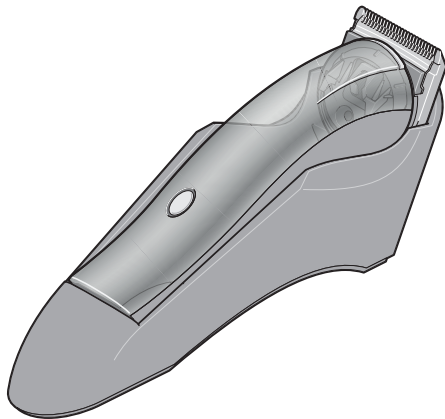

OC20

Deutsch (Original-Betriebsanleitung).....	2
English .....	14
Français .....	26
Español .....	40
Português .....	54
Italiano .....	66
Nederlands .....	78
Dansk .....	90
Svenska .....	102
Suomi .....	114
Norsk .....	126
Ελληνικά .....	138

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# 1. IMPORTANT SAFEGUARDS

## DANGER

When using electrical appliances, basic safety precautions should always be followed including the following:

### READ ALL INSTRUCTIONS BEFORE USING

- ⦿ Close supervision is necessary when this appliance is used by, on, or near children or individuals with certain disabilities.

- ⦿ Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- ⦿ Never operate this appliance if it is not working properly or if it has been dropped or damaged. Return the appliance to a service center for examination and repair.
- ⦿ Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.
- ⦿ Do not use this appliance with a damaged or broken comb, as it has the potential to result in injury.
- ⦿ Only use the included original power adapter for AC power.

## SAVE THESE INSTRUCTIONS

### 2 Safety

Please read the user instructions carefully, especially the chapter on “Safety”, **before** using the appliance. This will protect you from possible dangers and the appliance from damage due to incorrect operation.

Keep these user instructions in a safe place for future reference. If you pass on the appliance to someone else, be sure to hand over these user instructions as well. The user instructions are a part of the product.

The type plate is on the underside of the appliance.

### Explanation of warnings

If necessary, the following warnings will be used in these user instructions:

#### DANGER!

In combination with the word “Danger”, this symbol indicates a high risk. Ignoring the warning will result in death or serious injury.

**WARNING!**

In combination with the word "Warning", this symbol indicates a moderate risk. Ignoring the warning may result in death or serious injury.

**CAUTION!**

In combination with the word "Caution", this symbol indicates a low risk. Ignoring the warning may result in minor or moderate injuries.

**ATTENTION!**

Possible risk of damage to the device.

**NOTE:** Circumstances and specifics that must be observed when handling the device.

**Intended use**

- ⦿ This hair clipper for animals may only be used for cutting hair of small animals (e.g. dogs and cats).
- ⦿ The appliance is designed for indoor use only!

**Foreseeable misuse**

- ⦿ Use the hair clipper for animals only to cut hair of small animals. Under no circumstances may this appliance be used on people.
- ⦿ Do not use the appliance on wet hair.

**Instructions for safe operation**

- ⦿ The appliance is designed for commercial use.
- ⦿ The device is only to be used for cutting hair of small animals.

- ⦿ This device contains a battery that cannot be replaced.

- ⦿ Only use the detachable power adapter that came with this device to charge the battery.

- ⦿ **DANGER!** Do not use this appliance close to bathtubs, showers, washbasins or other receptacles containing water.

- ⦿ **WARNING!** Keep the appliance dry.

- ⦿ This appliance can be used by children from the age of 8 and people with restricted physical, sensory or intellectual abilities or people without adequate experience and/or understanding, if they are supervised or instructed on how to use this appliance safely and if they are aware of the resulting dangers. Children must not play with the appliance. Cleaning and user

maintenance must not be performed by children without supervision.

- ⊙ Use only the original mains plug adapters.
- ⊙ If the power adapter (including the power cord) is damaged, it must be replaced by the manufacturer or his customer service in order to avoid hazards.
- ⊙ Observe the notes on cleaning and care of the appliance (see “Cleaning and care” on page 21).
- ⊙ After an extended use, the cutter head may become hot.

**DANGER as small parts may be swallowed**

- ⊙ The lid of the oil bottle is a small part that could be swallowed. Keep away from children under 36 months of age.

**DANGER for children**

- ⊙ Packing materials are not children's toys. Children should not be allowed to play with the plastic bags: there is a risk of suffocation.

**DANGER! Risk of electric shock due to moisture**

The following safety instructions apply to the **connected charging station, and to the appliance when it is being operated with the power supply unit.**

- ⊙ Do not use this appliance or charging station close to baths, showers, wash-basins or other containers containing water.
- ⊙ Never place the appliance anywhere that it could fall into water while plugged in.
- ⊙ If the appliance has fallen into water, pull out the power supply unit immediately, and only then remove the appliance from the water. Do not use the appliance again, and have it checked by a specialist workshop.
- ⊙ Protect the appliance from moisture, water drops and water splashes.
- ⊙ The appliance, the connecting cable, the charging station and the power supply unit must never be immersed in water or other liquids.
- ⊙ If liquid gets into the appliance, remove the power supply unit immediately. Have the appliance checked before reusing.
- ⊙ Never touch the appliance with wet hands.
- ⊙ Do not rinse it under running water.
- ⊙ After use, pull out the power supply unit. The proximity of water represents a hazard even when the trimmer is not being charged.
- ⊙ Installation of a residual current device (FI/RCD) with a rated residual operating current of no more than 30 mA into the electrical circuit is recommended

to provide additional protection. Ask your electrician for advice. The installation should only be carried out by a qualified electrician.

### **DANGER! Risk of electric shock**

The following safety instructions apply to the **connected charging station, and to the appliance when it is being operated with the power supply unit.**

- ⊙ Check the appliance and the connecting cable regularly for damage. Do not use the appliance if the appliance or the connecting cable has any visible damage, or if the appliance has previously been dropped.
- ⊙ Only connect the power supply unit to a properly installed and easily accessible power socket, whose voltage corresponds to the specifications on the type plate. The wall socket must still be easily accessible after the appliance is plugged in.
- ⊙ Lay the connecting cable such that no one can step on it, get caught on it, or trip over it.
- ⊙ Please be careful not to damage the connecting cable:
  - Keep the connecting cable away from hot surfaces and sharp edges.
  - Ensure that the connecting cable cannot be trapped or crushed.
  - Do not wrap the connecting cable around the appliance. The cable may break.
- ⊙ To pull the power supply unit out of the power socket, always pull on the power supply unit, and never on the connecting cable.
- ⊙ Never attempt to remove dust and foreign objects from inside the appliance using pointed objects.

- ⊙ Pull the power supply unit out of the power socket, ...
  - after every use,
  - after each charging process (does not apply to charging station),
  - if there is a fault,
  - before oiling the appliance,
  - before you clean the appliance and
  - during thunderstorms.
- ⊙ Ensure that the power supply unit cannot be plugged in again during cleaning.
- ⊙ In order to avoid any hazards, do not make modifications to the appliance. You also must not replace the connecting cable yourself. Repairs must be carried out exclusively by your authorised customer service centre.
- ⊙ Use only original spare parts, which you can order from the customer service centre (see the back of these user instructions). If the connecting cable is damaged or crushed, both it and the power supply unit must be completely replaced.

### **WARNING! Risk from batteries**

- ⊙ Protect the battery against mechanical damages. **Fire hazard!**
- ⊙ Do not expose the appliance to direct sunshine or heat. The ambient temperature should not be lower than 0 °C or higher than +40 °C.
- ⊙ The charge contacts on the appliance must not be connected by metallic objects.
- ⊙ Charge the appliance's built-in battery exclusively using the original accessory parts.
- ⊙ The appliance must only be operated with the original  C20 power supply unit.

- ⊙ If the electrolyte solution leaks out of the battery, avoid contact with eyes, mucous membranes and skin. Rinse immediately with plenty of clean water and consult a doctor. The electrolyte solution can cause irritation.
- ⊙ The appliance contains a lithium ion battery.
  - This appliance must not be opened!

### **WARNING! Risk of injury from cutting**

- ⊙ Do not use the appliance with a damaged or fractured cutter head.
- ⊙ The cutter head must not be disassembled.
- ⊙ While the appliance is switched on, do not touch the cutter head.
- ⊙ The appliance must be switched off:
  - when mounting or removing an attachment comb.
  - when you put down the appliance, e.g. when you interrupt the cutting process briefly.
  - before removing or mounting the cutter head.
  - before cleaning the appliance.
- ⊙ Use only the original attachment combs.
- ⊙ Do not use a damaged attachment comb.

### **WARNING before working on or with animals**

- ⊙ Working on animals should only be performed by people who are experienced in cutting animal hair or who work under the guidance of an experienced person.
- ⊙ Do not use the device on a restless or aggressive animal.
- ⊙ Animals must be calmed down and, if necessary, fixed before cutting.

- ⊙ Make sure that your workplace is quiet and always work with the necessary caution.
- ⊙ Depending on the type of animal, protective gloves may be required as personal protective equipment.
- ⊙ Stop working immediately if the animal behaves conspicuously or becomes aggressive.

### **ATTENTION! Risk of material damage**

- ⊙ The cutter head is a precision part. Protect it against damage.
- ⊙ In order to avoid increased wear of the cutter head and motor, the cutter head must be cleaned and oiled regularly (see "Cleaning and care" on page 21).
- ⊙ Do not clean using sharp objects or abrasive cleaning agents.
- ⊙ The battery charger is fitted with non-slip plastic feet. As furniture is coated with a wide array of varnishes and synthetics, and is also treated with different care products, it cannot be fully ruled out that some of these materials contain components that could attack and soften the non-slip plastic feet. If necessary, place a non-slip mat under the appliance.
- ⊙ Protect the appliance against heat (e.g. direct sunlight), chemical substances (e.g. perm agents) and mechanical damage (e.g. by dropping).

## Specifications according to Machinery Directive 2006/42/EC

**A-weighted emission sound pressure level at workstations at a distance of 1 m,  $L_{pA} \leq 70$  dBA**

measured maximum noise emission value, including uncertainty of measurement according to EN ISO 4871. Determination of the values according to EN 60704-2-9 with reference to EN 60704-1

**Vibration total value to which the hand-arm system is subjected,  $a_h < 3.96$  m/s<sup>2</sup>**

measured maximum vibration total value including uncertainty of measurement. Determination of the values according to EN ISO 5349-1/2

## Designation of the Parts

- |   |   |
|---|---|
| 1 On/Off switch with LED for indication of charging state | 9 Charge contacts in charging station                     |
| 2 Cutting-length lever                                    | 10 Port in the appliance                                  |
| 3 Cutter head   | 11 Charge contacts of the appliance                       |
| 4 Cleaning brush  | 12 Release key  |
| 5 Oil bottle  | 13 Mains plug adapter (replaceable, figure: EURO version) |
| 6 Comb attachment (x5)                                    | 14 Connecting cable                                       |
| 7 Accessory holder  | 15 Power supply unit (model C20)                          |
| 8 Charging station  | 16 Port in the charging station                           |

## 3 Preparation

### 3.1 Changing the mains plug adapter


The appliance is supplied with a mains plug adapter 13. Other mains plug adapters can be ordered from the customer service centre (see service address on the back of these user instructions).

#### WARNING!

- ⊙ Use only the original mains plug adapters 13.
- ⊙ Pull the power supply unit 15 out of the power socket before you change the mains plug adapter 13.

**NOTE:** ensure that you do not bend the small pins on the power supply unit 15.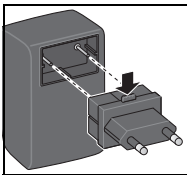

1. Hold down the release key 12 while pulling off the mounted mains plug adapter 13.
2. Insert the required mains plug adapter 13 into the power supply unit 15. Ensure that the release key 12 is located at the top and audibly clicks into place.

### 3.2 Setting up charging station and accessory holder

- Figure A: insert the accessory holder **7** sideways into the brackets below the charging station **8** (as shown). The accessory holder allows you to keep the comb attachments **6**, the oil bottle **5** and the cleaning brush **4** to hand at all times.
- Place the charging station on a level, non-slip surface.
- Lay the connecting cable **14** such that nobody can trip over it.
- Keep the connecting cable **14** away from hot surfaces.

### 3.3 Charging process

The appliance is not yet ready for operation when delivered.

**Before first use, the appliance should be charged for 2 hours.**

The charging time is then approx. 1 hour. "Overcharging" the appliance is not possible.

#### WARNING!

- ⊙ Before plugging in the power supply unit **15**, check that the voltage of the wall socket corresponds to the specifications on the type plate.

1. Plug the power supply unit **15** into a wall socket.
2. Figure C: insert the plug of the connecting cable **14** into the port in the charging station **16**.
3. Place the appliance into the charging station **8**. It positions itself automatically and starts charging. The LED in the On/Off switch **1** is lit continuously. The colour changes with the charging state from almost empty (orange) to fully charged (green). During charging, the LED will flash every couple of seconds.

**NOTE:** figure B: you can also charge the appliance by plugging the plug of the connecting cable **14** directly into the port in the appliance **10**.

### 3.4 Cable mode

You can also use the device in cable mode, e.g. when the battery is not charged.

#### WARNING!

- ⊙ Before plugging in the power supply unit **15**, check that the voltage of the wall socket corresponds to the specifications on the type plate.

1. Plug the power supply unit **15** into a wall socket.
2. Figure B: to do this, insert the plug of the connecting cable **14** directly into the port in the appliance **10**.

## 4 Operation

### 4.1 Switching On/Off

- To switch the appliance on, briefly press the On/Off switch **1** on top of the appliance. The appliance starts to operate and the LED of the On/Off switch is lit.
- To switch the appliance off, hold down the On/Off switch **1** on top of the appliance for a few seconds.

## 4.2 On/Off switch - LED colours

The colours of the On/Off switch **1** indicate the charging state of the battery during operation and charging:

Colour	Charging state
Red	The battery is almost empty.
Yellow	The battery is partially charged.
Green	The battery is full.

## 4.3 Adjusting the cutting length

**NOTE:** only use original accessories!

### Adjusting the cutter head

- The cutting length of the cutter head **3** is adjustable in 4 steps, from 0.5 to 2.5 mm.
- Use the cutting-length lever **2** on the underside of the appliance to set the desired cutting length.

### The attachment combs

- Figure D: switch off the trimmer and push the comb attachment **6** (4, 8, 12, 16, 20 mm) onto the cutter head **3**.
  - You can then cut hair with the trimmer as usual.
- You can achieve the best cutting results in combination with a comb attachment **6** when the cutting-length lever **2** is set to 1 (0.5 mm).

## 5 Cleaning and care

To ensure perfect functioning and full performance of your trimmer, clean and oil the appliance regularly.

### Cleaning and oiling the cutter head

If you use the appliance regularly, the cutter head **3** should be thoroughly cleaned and oiled **once a day**.

- Switch off the appliance.
- Remove the comb attachment **6**, if necessary.
- Remove any hair using the cleaning brush **4**.
- Figure E: oil the areas where the bottom and top cutting blades touch.

### Removing the cutter head

The cutter head **3** should additionally be removed **at regular intervals**, in order to carefully remove residual hair particles from the interior using the cleaning brush **4**.

1. Ensure that the appliance is switched off (LED in On/Off switch **1** is not lit).
2. **Removing:** figure F: press on the top of the cutter head **3** with your thumb to push it off the appliance.
3. Figure G: press the upper and lower blades apart slightly and use the cleaning brush **4** to remove the hair from the gap.
4. **Mounting:** figure H: place the cutter head **3** on at the bottom and swing it upwards until it clicks into place.

## Replacing the cutter head

The cutter head **3** is a wear part. Despite good care, the cutting performance will decrease at some point. Replace the cutter head when this happens. The cutter head can be ordered separately as a spare part from the customer service centre. The address and telephone number of your nearest customer service centre can be found on the back of these user instructions.

## Cleaning the appliance

- From time to time, wipe the appliance with a slightly moistened cloth, then rub dry.

## Cleaning the charge contacts

If the charge contacts on the appliance **11** or the charge contacts on the charging station **9** or the charge contacts on the plug of the connecting cable **14** are dirty, charging may be interrupted or obstructed.

- Clean the charge contacts with a piece of cotton wool or cloth dipped in alcohol or methylated spirit.
- Dry the charge contacts before using them again.

## Care of the battery

- Ensure that the appliance is switched off (LED in On/Off switch **1** is not lit) when placing it in the charging station **8**.

## 6 Service

Given the present state of the art, components of electrical appliances are subject to a certain degree of wear over time.

In order to ensure proper functioning and safety of the appliance, a regular check must be performed in accordance with the country specific legal provisions - though at least once every 2 years.

## 7 Technical data

Product	Type: OC 20
Mains adaptor Input Output	Type: C20 100-240 V; 50-60 Hz 6.5V $\equiv$ / 900 mA
Cutting length	adjustable: 0.5 - 2.4 mm 0.02 - 0.1 in
Dimensions (L x W x H)	175 x 46 x 50 mm 6.89 x 1.81 x 1.97 in
Weight (hand piece)	240 g / 0.53 lbs

## Power adapter

Published data	Value and accuracy	Unit
Ansmann AG Industriestraße 10 97959 Assamstadt Germany Reg. Amtsgericht: Ulm HRB B721168	–	–
Model identification: SEH0650900V / C20	–	–

Published data	Value and accuracy	Unit
Input voltage	100 – 240	V
Input AC frequency	50 – 60	Hz
Output voltage	6.5	V
Output current	0.9	A
Output power	5.85	W
Average efficiency in operation	79.32	%
Efficiency at low load (10 %)	Not applicable	%
Power consumption at no load	0.07	W

Subject to technical modifications.

The pictures may differ in these instructions.  
The serial number can be found inside the housing when the cutter head is removed.

Sign for the UK conformity

Sign of Australian conformity

## 8 Disposal

The battery integrated into this appliance must not be treated as household waste. The appliance with the installed battery must be disposed of properly.

The product contains electronic components and / or batteries. Do not dispose of the appliance in your household waste, instead take it to a municipal collection site for recycling.

## 9 FCC hint

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**CAUTION:** To comply with the limits of the Class B digital device, pursuant to Part 15 of the FCC Rules, this device is to comply with Class B limits. All

peripherals must be shielded and grounded. Operation with non-certified peripherals or non-shielded cables may result in interference to radio or reception.

**MODIFICATION:** Any changes or modifications not expressly approved by the guarantee of this device could void the user's authority to operate the device.

## 10 Warranty

Dear customers,

We guarantee all our products for a period of twenty-four months from the date of purchase. This warranty applies to all defects resulting from material or manufacturing defects.

- In the unlikely event that a fault occurs in the device during the warranty period, we guarantee to replace the device by either repairing or exchanging it with another flawless device at our own discretion.
- These services shall only be rendered upon presentation of the original invoice and/or the receipt. Retain the receipt as an evidence of the purchase date.
- Warranty excludes damages caused through improper use and normal wear.
- Warranty commitment also lapses if the product is tampered with by unauthorised agencies or if third-party products are used for installation.
- Faults, which only influence the value and the usability of the product to a slight extent, are not included in warranty.
- Wear parts are excluded from warranty. Typical parts subject to regular wear and in this respect are not liable to our warranty, are in particular the battery and the knife head.
- Kindly contact the concerned service for warranty claims; service address is on the reverse side of this manual.