

GVP

General Vet Products

(Box of 10)

DARVALL Cozy™

Surgery Under Blanket

Medium & Large Dog for Anesthesia & Surgery


Effective

Thermodynamic

Solving Hypothermia

Rapid Response

Safe Warming

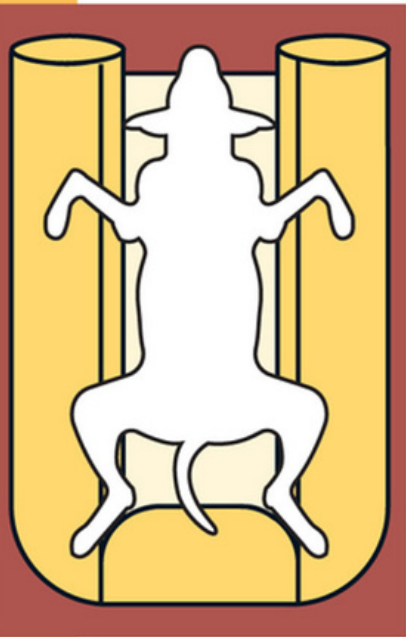

Specifications:

For Medium & Large Dogs

Blanket consumable

Inflation size:

0.9m (36")L x 0.4m (16")W

Inflation tube 10cm (4") diam.

*Designed for use with
The Darvall Cozy™ Heater

HEPA filter antimicrobial 0.2um
replace / 500 hours

Designed by Vets for Veterinary Use Only
Patents granted & pending

Guide for use:

- Compatible with most warm air heaters*
- Place blanket under the animal, white side up as indicated on the inlet port
- Insert the heater hose nozzle into one of the blanket inlet ports
- Position the animal either dorsal or lateral; do not occlude inflation tube along the inlet side
- Inflate blanket after the animal is draped for surgery
- Follow the heater's instructions to commence warming
- Monitor the animal's temperature frequently; once it reaches 37°C (98.6°F), set heater to 38°C (100°F)
- Replace blanket if hair covered, wet or soiled

GVP



1300 838 438

sales@generalvetproducts.com.au

www.generalvetproducts.com.au

