


# GVP

General Vet Products

# DARVALL Cozy™

## Surgery & Dentistry Blanket

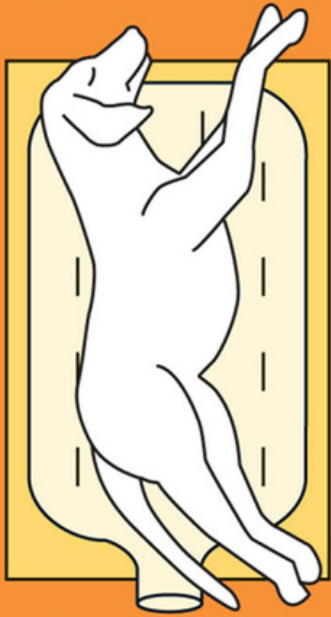

### Medium & Large Dog Orthopedic Surgery Over-Blanket Dentistry Under-Blanket

- Effective
- Thermodynamic
- Solving Hypothermia
- Rapid Response
- Safe Warming

#### Specifications:

For Medium & Large Dogs

Blanket consumable

Inflation size:

1.1m(43")L x 0.65m(26")W

Inflation Tubes 9cm(3.5")diam.

\*Designed for use with  
The Darvall Cozy™ Heater

HEPA filter antimicrobial 0.2um  
replace / 500 hours

Designed by Vets for Veterinary Use Only  
Patents granted & pending

#### Guide for use:

- Compatible with most warm air heaters\*
- For Surgery Place blanket over animal, white side to the animal as indicated on inlet port
- Inflate blanket after the animal is draped for surgery
- For Dentistry place blanket underneath with white side to the animal, inlet port at the tail end
- Insert the heater hose nozzle into the blanket inlet port
- Do not occlude the inflation tube at the blanket inlet
- Follow the heater's instructions to commence warming
- Monitor the animal's temperature frequently; once it reaches 37°C (98.6°F), set heater to 38°C (100°F)
- Replace the blanket if hair covered, wet or soiled

# GVP



1300 838 438

[sales@generalvetproducts.com.au](mailto:sales@generalvetproducts.com.au)

[www.generalvetproducts.com.au](http://www.generalvetproducts.com.au)

