


# DARVALL Cozy™

## Kennel Over Blanket

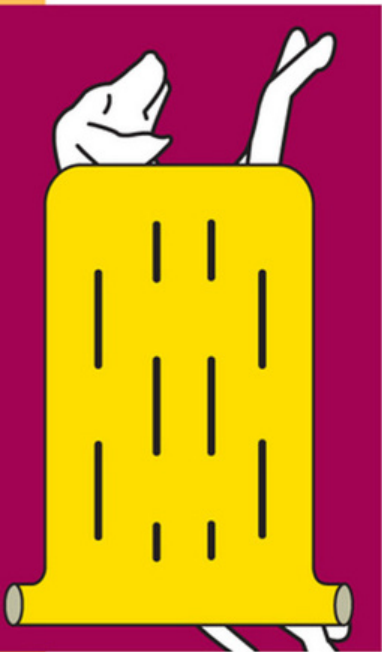

### Medium & Large Dog for Kennel or ICU Pre-Warming & Recovery

- Effective
- Thermodynamic
- Solving Hypothermia
- Rapid Response
- Safe Warming

#### Guide for use:

- Compatible with most warm air heaters\*
- Position the animal in the kennel
- Place the blanket over, with white side to animal
- Insert heater hose nozzle into the outside of the kennel door adapter (Part 5366 or 7882) & connect one of the blanket inlet ports to the inside of the adapter
- Do not occlude the inflation tube at the blanket inlet when closing the kennel door
- Follow the heater's instructions to commence warming
- Monitor the animal's temperature frequently; once it reaches 37°C (98.6°F), set heater to 38°C (100°F)
- Replace blanket if hair covered, wet or soiled

#### Specifications:

For Medium & Large Dogs  
Blanket consumable  
Inflation size:  
1.0m (39")L x 0.65m (26")W  
Inflation tubes 9cm(3.5")diam.

Use with Darvall's  
Kennel Door Adapter

\*Designed for use with  
The Darvall Cozy™ Heater

HEPA filter antimicrobial 0.2um  
replace / 500 hours

Designed by Vets for Veterinary Use Only  
Patents granted & pending

# GVP



1300 838 438

[sales@generalvetproducts.com.au](mailto:sales@generalvetproducts.com.au)

[www.generalvetproducts.com.au](http://www.generalvetproducts.com.au)

